

## WARNING

Since you have taken the trouble to have an earimpression taken of your ears for your personal sleeves it is logical that these are for you and for you alone. Needless to say, there are clearly less smart people out there so we'll say it anyway...We were taught to share in primary school, but DON'T do this! Not with your earplugs anyway. Besides that they won't fit, it is also unhygienic and can cause all kinds of earwax exchange problems. Just don't do it!

### Geluidsniveau in dB

85 dB	83 dB	86 dB	89 dB	92 dB	100 dB	110 dB
8 uur	4 uur	2 uur	1 uur	30 min	5 min	Direct

Maximale blootstelling in tijd

## CUSTOM SLEEVES

Whooptidoo, you are now the proud owner of this incredibly genius piece of art created by ears4U. We understand that you are excited and want to start using them right away, but before you do, there are a few things that are good to know. So please read on...

To ensure that the sound quality remains optimal, it is advisable to have your ears checked regularly for blockage. Consult your doctor for this. The custom-made sleeves also require some maintenance. More about this in maintenance on the other side.



[www.ears4u.nl](http://www.ears4u.nl) | [info@ears4u.nl](mailto:info@ears4u.nl)

**CUSTOM SLEEVES**


## INSTALLING THE IN-EARS

Your ears4U sleeves are handmade with love and attention so that they can nestle nicely in your ear. For optimal comfort, use the appropriate sleeve on the correct ear, which means... The sleeve with the red dot goes on your right inear and the sleeve with the blue dot goes on your left inear.



## PLACING IN THE EARS

Grab the sleeve between your thumb and index finger. Look closely at the sleeve, make sure the dot is facing up. Now slowly, carefully and with love and attention bring the sleeve to the ear, turn it a quarter turn back and place it in the ear. This should feel comfortable and NOT painful in any way. If it is painful to wear the sleeve, please contact ears4U immediately.

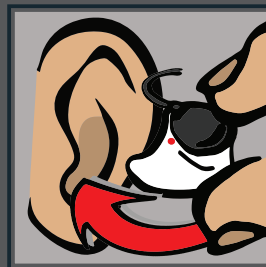

## REMOVING FROM THE EAR

When you have finished using the inears, do not unwittingly yank them out of your ear. But again with love and attention. Place your thumb at the bottom against the edge of the sleeve and gently push it up and remove the sleeve from your ear. The inears can simply remain on the sleeves.



## MAINTENANCE

Regularly check whether the sound openings of the sleeves are free of contamination and earwax. Any blockage can be removed with the supplied hook. The sleeves are provided with an antibacterial lacquer that can be cleaned with a damp cloth and possibly alcohol. If the sleeves are not on the inears, they can be rinsed under the tap.

